BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА
EN: USER MANUAL
DE:GEBRAUCHSANWEISUNG
ES: INSTRUCCIONES DE USO
GR: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
IT: ISTRUZIONI PER L'USO

NL: GEBRUIKERSHANDLEIDING
RO: INSTRUCȚIUNI DE UTILIZARE
РУ: ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
SR: UPUTSTVO ZA UPOTREBU
FR: NOTICE D'UTILISATION

CARA

D-111

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Уеб сайт: www.cangaroo-bg.com

EN

READ CAREFULLY AND KEEP FOR FUTURE REFERENCE! THE MANUAL CONTAINS IMPORTANT INFORMATION, RECOMMENDATIONS AND GUIDELINES FOR THE PRODUCT AND ITS SAFE USE.

The manual breast pump of Cangaroo provides many of the functions the busy moms love and is perfect for many reasons:

- The expression and storage of the breast milk, while you are away from Your baby.
- Relieving the tension in your breasts because of the large amount of breast milk. and facilitating the process of baby breast milk feeding.
- Maintaining your milk supply if you have to temporarily stop breastfeeding.
- Its compact and supplies the discreet usage in order you to take it anywhere with you and to express the breast milk in a convenient time for you.

The manual breast pump is equipped with soft massage pad, which is designed to give you the feeling of softness and warmth and to mimic the baby sucking, stimulating the faster flow of the breast milk – easy, comfortable and gentle. The breast pump is compact and easy to assemble, all of the parts can be washed in a dishwasher and it does not contain Bisphenol – A.

According to the health specialists the breast milk contains antibodies, which provide the protection of the baby from infections and allergies.

I. RULES AND RECOMMENDATION FOR SAFE USE

1. Use this product only for its intended purpose as described in this manual
2. This unit is not intended for usage by persons (including children) with reduced physical, sensory or mental capabilities or the lack of experience and knowledge unless they have been given supervision or instruction by a person responsible for their safety.
3. **Warning!** The manual breast pump is intended for repeated use only by one user.
4. **Warning!** Children should not play with the manual breast pump.
5. **Warning!** Before first usage check the manual breast pump for signs of damage. If you find such, do not use the product until the fault is removed. Do not try to modify or repair the product by yourself. For consultation contact an authorized service or the sales agent, from which you have bought the manual breast pump.
6. Do not use any accessories, which are not approved by the manufacturer and do not make any improvements or changes on the product as this may lead to its malfunction or injuries

to the person, who uses it.

7. Do not use spare parts, which are not the original ones supplied by the manufacturer or the importer. Using such, Your guarantee card will be invalid.

8. Do not use while you are pregnant as this may lead to stimulating the birth process.

9. Never use while you are sleepy or drowsing.

10. Do not leave it and do not keep the product in a place with too high room temperatures or at direct sunlight.

11. Be careful not to drop the product and to put in its opening or in the feeding bottle other objects or liquids.

12. If you are infected with Hepatitis A, Hepatitis B or HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.

13. Wash, rinse and sterilize all the parts of the breast pump after each use. Do not clean the part of the breast pump with strong detergents – alcohol based ones, such containing bleach or with abrasive parts.

14. Do not pump more than 5 minutes without a pause, if the milk does not express.

15. If the vacuum and pressure cause you discomfort or pain, stop using the breast pump by firstly putting finger between the breast and silicone massage pad and then remove the pump from your breast.

16. **WARNING!** This product is intended for personal usage and it is not intended for the usage of other mothers.

17. The breastmilk bottle in the set is an integral part of the breast pump. For the purposes of this product use only this bottle and only for the breast milk collected with this pump.

FOR YOUR CHILD'S SAFETY AND HEALTH

WARNING!

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Keep components not in use out of the reach of children.
- Throw away at the first signs of damage or weakness.
- Inspect carefully before each use. Pull the soother in all directions. Throw away at the first signs of damage or weakness.
- Do not leave the soother to direct sunlight or any other heat source, or in disinfectants (sterilizing solution) longer than recommended as this can weaken the teat.

II. BASIC PARTS OF THE MANUAL BREAST PUMP

SEE THE FIGURE PD:

Basic part of the breast pump	Material of which is made the part	Temperature resistance	Cold resistance
1. Dust cover	PP	120°C	-20°C
2. 3D massage pad	Silicone	200°C	-20°C
3. Connector	PP	120°C	-20°C

Basic part of the breast pump	Material of which is made the part	Temperature resistance	Cold resistance
4. Fastening cover	PP	120°C	-20°C
5. Silicone cylinder	Silicone	200°C	-20°C
6. Carry ring	PP	120°C	-20°C
7. Handle	ABS	80°C	-20°C
8. Valve	PP	120°C	-20°C
9. Breastmilk bottle (150ml)	PP	120°C	-20°C

III. ASSEMBLY OF THE MANUAL BREAST PUMP

Make sure that you have cleaned well and sterilized all the parts of the manual breast pump before starting its assembly. Wash your hands thoroughly! Follow exactly the instructions and sequence for assembling the product, described in the text and images hereafter attached.

- 1. Step 1:** Attach to the carry ring the silicone cylinder. Be careful not to tear or scratch the cylinder as this will affect the pumping vacuum. Put the silicone cylinder into the upper part of the pump body.
- 2. Step 2:** Press with fingers the top part of the cylinder to the upper part of the pump body to fix it well. It should fits tightly to the pump body. .
- 3. Step 3:** Attach the pump handle to the ring.
- 4. Step 4:** Put the cover over the cylinder and handle.
- 5. Step 5:** Attach the silicone massage pad to the pump body, as shown in figure 5.
- 6. Step 6:** Gently push its inner end as further as possible and make sure that it fits ideally to all sides of the pump body.
- 7. Step 7:** Fix the breastmilk bottle to the pump body so that the thread of the bottle coincides exactly with the thread of the bottom part of the pump connector.
- 8. Step 8:**The breast pump is ready for use. On Figure 8 is shown the right way to hold the breast pump when using it.
- 9. Step 9:** Put the protective cover if you will not be using the pump.

IV. METHOD OF USE

A. WHEN YOU CAN USE THE BREAST PUMP:

1. If it is possible.

- Only if you have sufficient milk secretion and normal time for breastfeeding.
- The use of the breast pump may begin usually two or four weeks after the birth of Your baby until you set a graphic of expressing the breast milk and the breastfeeding. Unless if Your doctor/ health advisor hasn't recommended you another way of expressing your breast milk.

2. In the following cases You can always use Your breast pump:

- If you have more breast milk, you can express some of it between meals in order to reduce the pain of the bulging breasts, and this way it is easier for the baby to suck.
- If your nipples pain or are cracking, you can express some milk for feeding until the nipples get back.
- If you need to separate from your baby and hope to continue breastfeeding after reunion, breast milk can be expressed periodically to maintain milk secretion.

B. HINTS:

1. It takes practice in order to use the breast pump the best way possible. So, it will take you at least several attempts before starting successfully to use the manual breast pump.
2. Choose a quiet time to use the pump – time, when no one will disturb you. If you are working again, it may be needed to express in the breaks.
3. You should find the optimal expressions for the day, for instance right before and after the first breastfeeding in the morning, when Your breasts are full or after breastfeeding, when the baby hasn't taken all from both of Your breasts.
4. The picture of Your child may help to stimulate the expression of the milk.

5. Sit in a comfortable chair, in a perpendicular position to the ground. Please, put a comfortable cushion behind your back before you start using the breast pump. Take a glass of water and place it near you, because you may feel thirsty during the pumping process.
6. Place a warm cloth on Your breasts for a few minutes before starting to express the breast milk as this may make the process easier and relieve the tension in the breasts.
7. Warmth and relaxation help for the easier expressing the milk, so before using the breast pump, you can take a bath or a shower.
8. Try to breastfeed the baby on one of the breasts and in the same time to use the pump on the other, or continue to pump right after you stop feeding the baby.
9. Changing the position of the pump on Your breast from time to time may lead to better stimulation of the secretion. Practice in order to find the best expression technic for You.
10. If the process is causing you breast pain, stop immediately and consult with a doctor at once.

11. Breast pump use errors:

12. The manual breast pump is an appliance, which helps the women after birth to calm the pain in their breasts and to prolong the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. For some women, which are mothers for the first time, the mammary gland usually is not smooth enough. A reason sometimes can be the congenital nipple inversion or milk water shortage. So, we suggest you before purchasing and using the pump, to consult with a doctor who could confirm that your mammary gland is smooth and that the breast milk is in normal condition. Do not buy and use the pump only as a device to unblock your milk ducts.

USAGE OF THE MANUAL BREAST PUMP:

1. Wash your hands well and make sure that Your breasts are clean. Express some milk, in order to check if the milk ducts are not blocked
2. The pump should be sterile and assembled.
3. Press the pump onto your breast:
 - Make sure that the nipple is in the middle in order the silicone massage pad to fit perfectly to Your breast so as to create a perfect vacuum (do not let air into).
 - Start carefully pushing the handle downwards until you feel a slight suction. After that leave the handle to return to the starting position. Repeat this step a few times fast in order to stimulate the expression.
 - Establish a slower rhythm by pushing the handle downwards and keeping it pushed at most 3 seconds, before leaving it to return in the initial position. Continue doing this until expressing. If your hand tires, try with the other one or leave the hand to relax onto your body and express from the other breast.
 - Make sure that the milk flows successfully and the process of pumping does not make you uncomfortable.
 - It is not necessary to push the handle completely downwards, but only to this position in which you feel comfortable.
4. Do not worry if the milk doesn't flow immediately, relax and continue pumping. If the process is causing you breast pains, stop immediately and consult with doctor. **If there is no result when pumping, please, do not continue! Try to use the pump in another time of the day.**
5. On average you should pump for 12 minutes in order to express 150 ml of milk. Please, keep in mind that the quantity of milk varies from woman to woman. If you find that you usually express more than 150 ml on session, then you should use bigger bottle. **Do not overfill the bottle, in order the milk not to leak outside it.**
6. After stopping the expression, carefully take off the pump from Your breast and unscrew the bottle with milk from the pump body. Clean the other used parts according to the instruction in section "Cleaning and sterilizing".

V. HOW TO STORE THE BREAST MILK

1. Only milk collected with a sterile pump should be stored for feeding Your baby.
2. The breast milk can be stored in bottle warmer by 40°C, so that you can feed Your baby within 4 hours.
3. When storing freshly expressed milk, please, seal the milk. The breast milk can be stored for 10 hours in room temperature of 19 - 25°C.
4. The expressed milk can be stored in fridge for maximum 48 hours (but not on the door of the fridge) or in the freezer for maximum 3 months. For this purpose you use the breast milk storage bags "**Cangaroo Care**".

5. NEVER REFREEZE THE BREAST MILK AGAIN.

6. Never add new milk to the already frozen one.

VI. FEEDING THE BABY WITH THE EXPRESSED MILK

1. You may feed the baby with the expressed milk from the bottle.
2. If you use frozen milk, you should leave it to thaw naturally in the fridge. If you need it urgently, place it in a bowl with hot water and thaw it. After thawing the milk can be stored in a fridge for 24 hours.
3. Pour the breast milk in a feeding bottle and place it in a bowl with hot water or in an electrical bottle warmer.
4. Fix the bottle ring with the sterilized feeding teat to the bottle.
5. Always check the breast milk temperature before giving it to the baby.
- 6. Always throw away the breast milk, which have left after feeding. Never refreeze.**
- 7. NEVER HEAT THE MILK IN A MICROWAVE AS THIS MAY DESTROY VALUABLE NUTRIENTS AND ANTIBODIES.** In addition, the uneven heating may cause hot spots in the liquid, which could cause burns to your baby's throat.
8. Never immerse a frozen milk container in boiling water as this may cause it to crack/ split.
9. Never put boiling water directly into a feeding bottle, allow to cool for around 20 minutes before filling.
10. The feeding bottle in the set is an integral part of the breast pump. For the purposes of this product use only this bottle and only for the breast milk collected with this pump. Not to be used for mixing formula milk.

VII. CLEAN AND STERILIZE THE MANUAL BREAST PUMP

Clean and sterilize Your manual breast pump before using it for the first time and after every usage following the listed instructions.

1. Disassemble completely the manual breast pump.
- 2. CAREFULLY** take off and clean the silicone valve. If it gets damaged, the vacuum will not be able to be created and the milk will not be expressed.
3. Wash all the parts with small amount of hot water and with a little bit of not aggressive detergent and wash them thoroughly. You can also wash the parts in dishwasher only on the top rack. In order to wash the valve, you can rub it gently between your fingers with some liquid detergent. Do not put any objects in the valve as this may lead to damages.
4. Sterilize all the parts in steam sterilizer.
5. For all the breastfeeding moms, which breastfeed every 2 – 3 hours, we do not recommend to sterilize after each use.

The excessive disinfection and cleansing can easily lead to premature aging and reduce the life of the breast pump. It is recommended to thoroughly clean the stains and sterilize once a day.

6. IMPORTANT! If you use the breast pump as a device for storing the breast milk, you will have to take care for the disinfection of all the washable parts of the pump, otherwise the milk may go bad, as it is a product difficult to store.

VIII. STORING THE BREAST MILK PUMP

Keep the manual breast pump away from direct sunlight, as the continued exposure may cause discoloration. Store the manual breast pump and its accessories in safe dry place, away from the reach of children.

IX. TROUBLESHOOTING GUIDE

1. Lack of suction:

- Check if you have put all the parts correctly and if the silicone valve is placed tightly to the pump body opening.
- Make sure that the silicone massage pad is correctly placed and fits to the pump body and a perfect seal is created.

2. The breast milk cannot be expressed:

- Please, make sure that the breast pump is correctly assembled and that vacuum is created.
- Relax and try again.

3. Pain in the breast area when using the pump:

• Stop using the manual breast and consult with Your professional advisor for breastfeeding. If the pump is not detaching easily from the breast, you can break the vacuum, by putting a finger between the breast and massaging pad or between the breast and the pump funnel.

4. The breast milk is leaking down the massage pad:

• Take out, place and reattach the massage pad, in order to make sure that it is firmly fixed to the central down part of the funnel.

• Try to lean a little bit forward

5. Cracking and discoloration of the pump/ feeding bottle parts:

• Avoid any contact with antibacterial detergents or such with abrasive parts as they may lead to damage of the plastic parts. The combination of detergents, cleansers, sterilizing solution, mild water, softened water and variable temperatures in specific cases may lead to cracking of the plastic. If this occurs, please, do not use the product. The pump body and the breast milk storage container is safe to be cleaned in dishwasher, but the food, left on the rest of the dishes may cause their darkening. Follow the cleaning instructions exactly. Keep the product away from direct sunlight, as the prolonged exposure may lead to discoloration of the plastic parts.

6. Lost or broken parts:

• For spare parts, please, contact authorized service center or the trade agent, from which you have bought the product. Do not use such which are not approved or not supplied from the manufacturer.

• Do not try to repair by yourself or modify the product, because this may endanger Your health and the one of Your child.

X. CONFORMITY WITH EUROPEAN STANDARDS AND DIRECTIVES

This product has been designed and produced in compliance with the requirements of the European Directives: Directive 2001/95 / EC on General Product Safety; Directive 2011/8/EU on the restriction of use of Bisphenol A in plastic infant feeding bottles; Regulation 10/2011/EU on plastic materials and articles intended to come into contact with food. **The product complies with the requirements of the following European standards:** EN 14350:2020 „Child care articles. Drinking equipment. Safety requirements and test methods“; EN 71-3:2019+A1:2021 „Safety of toys. Part 3: Migration of certain elements“.

Manufactured for CANGAROO

Manufacturer and Importer: Moni Trade LTD

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