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➤ **IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE!**

➤ **THESE INSTRUCTIONS CONTAIN IMPORTANT INFORMATION, GUIDELINES AND RECOMMENDATIONS FOR THE PRODUCT AND ITS SAFE USAGE.**

➤ Congratulations on the purchase of this electric pump. This product is an electric breast pump with different levels of control of the suction power, unique valve with gentle suction and soft massage silicone gel pad.

➤ The Electric breast pump complies with EU standards EN 55014-1/ 2; EN 61000-3-2/ 3-3; EN 60335-1; EN 62233:2008.

➤ It provides many of the features which the busy mothers love and is perfect for many reasons:

1. Expressing and collecting breast milk while you are away from your baby, for example when you are at work.
2. Relieving the tension in your breasts caused by the large amount of breast milk. For example, when your baby's feeding habits change because it has started eating more solids.
3. Maintaining your milk supply if you have to temporarily stop breastfeeding. For example, to express the unsuitable milk while you are undergoing medical treatment.

➤ The Cangaroo electric breast pump has the following features:

1. Massage cycle, so called stimulation – 5 Levels.
2. Suction function – 9 Levels.
3. Combined function massage with suction. It is with 9 levels and is as follows: 3 times massage and 1 suction.
4. Quiet motor providing discretion.
5. Light and easy to carry.
6. Easy to clean.

I. RECOMMENDATIONS AND WARNINGS FOR THE SAFE USAGE OF THE PRODUCT

READ CAREFULLY ALL INSTRUCTIONS BEFORE USING YOUR CANGAROO ELECTRIC BREAST PUMP!

1. The appliance is not intended for use by children. Keep the appliance and its adaptor out of reach of children.

2. Children shall not play with the appliance.

3. This appliance **SHOULD NOT** be used by children! The product can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and if they understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children.

4. Use this appliance only for its intended purpose as described in this manual.

5. WARNING: To reduce the risk of electrocution:

- Do not use while bathing or showering.
- Do not place or store this product where it can fall or be pushed into a bathtub or sink.
- Do not place or drop into water or other liquid.
- Do not reach after a product that has fallen into water. Unplug from the wall socket immediately.
- Close supervision is necessary when the product is used near children and disabled persons.
- Never operate this product if it has a damaged adapter or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
- Keep the adaptor away from heat sources.
- Make sure the voltage of the AC adaptor is compatible with the power source.
- Only use the AC adaptor which is included in the set. If any damage of the adaptor occurs, contact the manufacturer or the sales agent, from whom you have purchased the product.

6. Do not use attachments, which are not recommended by the manufacturer and do not make any improvements or changes in the product as this may lead to its malfunction or injuries to those who use it.

- 7.Never use just before falling asleep or if you are feeling drowsy.
- 8.Be careful not to drop the product and do not insert any other objects into the opening or tubing.
- 9.If you are infected with Hepatitis A, Hepatitis B or you are HIV positive (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
- 10.Do not drive a motor vehicle while using the electric breast pump.
- 11.Do not attempt to modify or repair the product by yourself when damage occurs. Instead you must contact an authorized service or the sales agent, from whom you have purchased the electric breast pump.
- 12.This appliance is intended for repeated use only by one user.
- 13.Do not allow children or pets to play with the product, its adaptor or the accessories included in the set.
- 14.Before each use inspect the pump and the adaptor for signs of damage. Do not use them if you find any damage.
- 15.Do not use spare parts which are not the original ones supplied by the manufacturer or importer.
- 16.Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the silicone pad with your finger, then remove the pump from your breast.
- 17.Never use while pregnant as pumping can induce labor.
- 18.Do not leave and do not store the product in a place with very high room temperature or under direct sunlight.
- 19.The product should not be left unattended when plugged into the electrical wall socket.
- 20.Before starting the breast pump, first connect the adapter to the electric breast pump, then securely connect the adaptor to the electric socket.
- 21.Wash, rinse and sterilize the appropriate pump parts before each use.
- 22.Never wash and sterilize the electrical parts of the product – the motor of the pump and the AC adaptor in order not to cause permanent damage to the product.

II. PARTS

- Figure PD: BL – bottle cap; SN – Silicone nipple; BC – Bottle ring; DS – Safety cover/ pump stand; MP – Silicone massage pad; AH – Air hole; PB – Pump body; BB – Baby milk bottle; V – Valve; P – Silicone piston; C – Pump cover; AT – connecting tube; PM – pumping mechanism; ATI – Connecting tube interface;
- Figure P - L:Levels; M:Massage; S:Suction; L-:Level down; L+:Level down; WT:Working time; B:Battery charge level;

III. CHARACTERISTICS

Type	Parameters
Item Number	XN-D258
Working Voltage	3.7V
Working Power	5W
Rated Input Voltage	5V
Rated Input Current	1A

Type	Parameters
Battery capacity	1400mAh
Adapter	AC100V – 240V; DC 5V, 1A
Time	120 minutes
Using time after full charge	160 minutes
Display	LED
Control	Touchscreen panel
Product size	Pumping mechanism: 118 x 77 x 39 mm Bottle + Pump body: 185 x 89 x 89 mm
Materials	PP – Safety Cover (stand) Pump cover, Bottle cap, Bottle ring, Pump body, Baby bottle Silicone – Valve, Silicone piston, Nipple, Silicone massage pad

IV. BATTERY CHARGE

1. Before first time use charge the battery completely.
2. The supplied adapter is with type C plug.

3. The breast pump is with a built-in battery 3.7V/ 1400 mAh.

4. Charging:

-Connect the adapter to the charging jack of the pumping mechanism.

-Insert the plug into electrical outlet of the mains supply of your home.

-We recommend you to leave the breast pump to charge completely before next use in order to prolong its battery life.

5. Use only the charger provided with the breast pump. Do not use chargers from other products.

6. Charge the battery on time. Charge as soon as the battery indicator is with 1 bar or the indicator starts flashing.

7. After the battery is fully charged the indicator will stop flashing.

8. If you will not use the product for a long time, charge the battery completely before storing it. Then charge the battery at least once a month to prolong its life.

9. If the battery is damaged or cannot be charged anymore, do not try to open the product by yourselves. For consultation and repair contact the shop from where you bought the product or the importer/ distributor.

10. The battery is recyclable. If the product is damaged and cannot be used any longer, take it to an official collection point for electronic products' waste, batteries, chargers. DO NOT throw the product away with the household waste. Follow your country's rules for the separate collection of electrical and electronic products and rechargeable batteries. Dispose of waste of this type only at the designated places. This will be important and significant contribution for the environmental protection and the reduction of the harmful effects of the substances contained in batteries and electronic components on the environment and human health.

V. ASSEMBLY OF THE ELECTRIC BREAST PUMP

ATTENTION! NEVER STERILIZE OR IMMEDIATE THE PUMP MECHANISM AND TUBE CONNECTING IT TO THE PUMP BODY - PART AT (See Figure PD). Follow the product assembly figures to ensure proper assembly.

1. Step 1: Place the silicone valve at the lower end of the pump body. Make sure that it is properly fixed around the ends of the pump body opening intended for it. Important! If the valve is loose or not installed correctly, it will not be possible to create a sufficient vacuum !

2. Step 2: Fix the feeding bottle to the pump body by turning the bottle gently clockwise so that the thread of the bottle coincides exactly with the thread of the pump body

3. Step 3: Place the piston in the upper part of the pump body. Make sure that it is properly fixed around the ends of the pump body opening intended for it.

4. Step 4: Put the protective cover on the upper side of the pump body. Press and turn clockwise to fit in the correct position.

5. Step 5: With clean hands, insert the silicone nozzle into the pump body. As you need to make sure it is placed tightly. Tip: If you are not going to use the breast pump, fit the protective cover.

6. Step 6: Insert one side of the connecting tube into the jack the pumping mechanism. Then insert the other side of the tube into the pump body. Connect the second part of the tube to the second pump body. Now you can use the breast pump.

VI. USE

- Wash your hands thoroughly and make sure your breasts are clean. Express some milk to check that your milk ducts are not blocked. The pump must be sterile and assembled.
- Sit down and relax in a comfortable chair, leaning forward slightly (Use a soft and comfortable pillow as a backrest).
- Put the pump on your breast and start the pump motor.

A. Massage (Stimulation) – The button to activate this function is ♥ :

This feature helps to stimulate the flow of breast milk. We recommend that you use this option first and then switch to suction mode. The massage mode mimics breastfeeding a baby, not too hard, and stimulates the flow of milk. The pumping mode also mimics baby sucking, but is stronger than the first mode, more even and with great efficiency. Press the massage button. You can adjust the stimulation levels with the "<" and ">" buttons. They are a total of 5.

B. Suction – The button to activate this function is ♡ :

Press the mode dial and activate the squeeze function. You can adjust the suction levels with the "<" and ">" buttons. They are a total of 9.

C. Combined function massage with suction:

Press the suction button and then the massage button. This way you activate the combined function. It is with 9 levels and proceeds as follows: 3 times massage and 1 time suction. You can adjust the levels with the "<" and ">" buttons.

D. Memory function: Turn off the pumping mechanism and automatically will be remembered the last level of massage that you used. Even if you have used the suction function, right before turning off the mechanism. When you turn on the mechanism again, the last saved massage level will automatically switch on.

E. Tips:

- If there is no result during pumping, do not continue! Try using the pump at another time of the day.
- On average, you have to pump for 8 minutes to drain 120 ml of milk. Keep in mind that the amount of milk varies from woman to woman. If you find that you usually squeeze more than 120 ml per session, then you should use a larger feeding bottle.
- Do not overfill the milk bottle to prevent milk from leaking out of the bottle.**
- When you stop using the pump, switch it off from the ON / OFF button. Then remove the tube from the back of the pump body. Remove the pump body from the bottle. Close the bottle with the ring and the pacifier and place a cap on top to keep it sterile.

VII. WHEN TO USE YOUR BREAST PUMP

1. **If it is possible.** Only if you have sufficient milk secretion and normal time for breast feeding. The use of the electric breast pump may begin usually two or four weeks after the birth of your baby.
2. **In the following cases you can always use your breast pump:**
 - When breast milk must be provided to your baby, but you are in the hospital.
 - If you have more breast milk, you can express some of it between meals in order to reduce the pain of the bulging breasts, and this way it is easier for the baby to suck.
 - If your nipples pain or are cracking, you can express some milk for feeding until the nipples get back.
- 3.If you need to separate with your baby and hope to continue breastfeeding after reunion, breast milk can be expressed periodically to maintain milk secretion.

VIII. HINTS

1. It takes practice in order to use the breast pump the best possible way. So it will take you at least several attempts before starting successfully to use the electric breast pump.
Choose a quiet time to use the pump – time, when no one will disturb you.
2. Sit in a comfortable chair, in a perpendicular position to the ground. Put a comfortable cushion behind your back before you start using the breast pump. Take a glass of water and place it near you, because you may feel thirsty during the pumping process.
3. Place a warm cloth on your breasts for a few minutes before starting to express the milk as this may facilitate the process and relieve the tension in the breasts.
4. Warmth and relaxation can help for the easier pumping of the breast milk, so before using the pump, you might like to take a bath or a shower.
5. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after the feed.
6. Repositioning the pump on your breast from time to time can help stimulate the secretion. Practice in order to find the best pumping technique for you. If the process is causing you breast pain, stop immediately and consult with a doctor at once.
7. Make sure that you have not blocked or covered the air opening of the pump body.

8. Breast pump use errors:

The electric pump is an apparatus, which helps postpartum women to ease the pain and to extend the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. For some women, which are mothers for the first time, the mammary gland usually is not smooth enough. A reason sometimes can be the congenital nipple inversion or milk water shortage. So, we suggest you before purchasing and using the pump, to consult with a doctor who could confirm that your mammary gland is smooth and that the breast milk is in normal condition. Do not buy and use the pump only as a device to unblock your milk ducts.

IX. HOW TO STORE THE BREAST MILK

1. Only breast milk that has been collected using a sterile breast pump can be stored to feed your baby.
 2. The milk can be stored in a thermos at 40 ° C, so you can feed your baby every 4 hours.
 3. If you store freshly expressed milk, seal the milk. The milk can be stored for 10 hours at room temperature at 19 - 25 ° C.
 4. The expressed milk can be stored in a refrigerator for a maximum of 48 hours (but not on the refrigerator door) or in a freezer for a maximum of 3 months.
 5. Frozen milk must first be left in the refrigerator to thaw naturally. If you need it urgently, put it in a bowl of warm water and thaw it. After thawing, the milk can be stored in the refrigerator for 24 hours.
- 6. NEVER FREEZE MILK AGAIN.**
7. Frozen milk can be gradually warmed in warm water or placed in a baby bottle warmer.
- 8. NEVER WARM YOUR BREAST IN THE MICROWAVE, BECAUSE YOU CAN DESTROY VITAL NUTRIENTS AND ANTIBODIES THIS WAY.** Also, uneven heating can lead to too hot areas of breast milk, which can cause burns to your baby's upper digestive tract. Never place a frozen bottle of milk in boiling water, as this may cause the bottle to burst / break. Never pour boiling water directly into a feeding bottle. You should allow the water to cool for about 20 minutes before filling the bottle.

X. CLEANING AND STERILIZATION OF THE PUMP

Important: NEVER CLEAN WITH WATER or STERILIZE the pumping mechanism (PM) and the connecting hose (AT). You can only clean it with a soft dry cloth.

Clean and sterilize your electric pump before first use and after each use, following the instructions below.

• **Sterilization:**

1. Remove the tube (AT) from the pump mechanism (PM) and set it aside. Separate all other parts: DS, MP, PB, P, C, V, BB, BC, SN, BL, then wash them with warm soapy water and dry them. Do not use antibacterial or corrosive cleaners.
2. Sterilize **ONLY** the following parts DS, MP, PB, P, C, V, BB, BC, SN, BL in a steam sterilizer or by boiling for 5 minutes.

❖ **RECOMMENDATIONS FOR CLEANING**

1. Use a steam sterilizer. Boiling the parts is not recommended. The first reason is that the product will be easily damaged if improper sterilization is applied. Second, this method of sterilization cannot achieve full effect.
 2. If you do decide to use boiling as a sterilization method, pour more water into the container so that the parts are completely covered with water. You must not leave the product unattended during the sterilization process. Do not boil for too long - maximum 10 minutes.
 3. For breastfeeding mothers who breastfeed every 2-3 hours, we do not recommend sterilizing after each use. Excessive sterilization can easily lead to premature aging of parts and shorten the life of the breast pump. Thorough cleaning of stains and sterilization once a day is recommended.
 4. Store the parts in a clean container until the next use.
- **IMPORTANT!** If you use the breast pump as a means of storing milk, you must take care to disinfect all washable parts of the pump, because otherwise, the milk can easily spoil, as it is a product that is difficult to store.
 - **ATTENTION!** Before use, make sure that the pump body, valve, piston and cover are dry.
 - **DO NOT STERILIZE THE TUBE – PART AT, to avoid accidental ingress of water into the pump motor.**
 - If you use a steam sterilizer to clean the pump parts, be careful that they do not come into contact with the heating plate of the sterilizer.
 - If you use boiling as a sterilization method, make sure that the parts do not come into contact with the bottom of the container in which you are boiling.

XI. TROUBLESHOOTING GUIDE

1. **Lack of suction:**

- Check if you have assembled all parts correctly and if the valve is positioned correctly.
- Make sure that the silicone massage pad is fitted securely and a perfect seal is created.
- Check if you have placed the silicone valve firmly to the opening of the pump body.
- Check whether there are any damaged parts.
- Check if the battery is down. Charge fully before use. Do not use during charging to prolong the battery life.

2. The suction is too low.

2. No milk is being expressed:

•Check if the silicone piston, the valve, the massage pad and the connecting tube are correctly attached.

•**Advice:** Adjust the suction level from low to high gradually depending on your need. The most suitable level is the one, with which the suction is painless. **The following is NOT true: the stronger – the better.** If you use too strong suction, this may harm your breast and cause pain and discomfort. So choose the level your feeling best.

3.Small quantity of expressed milk:

•The rate of breast milk production is different and strictly individual for each woman. It depends on many factors such as the mammary gland structure, the level of patency of mammary gland, your psychological and physical condition and etc. Sometimes the small quantity of expressed milk is not due to a problem with the breast milk pump but with your current physical and psychological health.

•**Tip:** To stimulate the flow of breast milk, apply a warm compress to your breasts for a few minutes before you start pumping. Then use the massage function for about 5 minutes and switch to suction. You can also massage your breasts by hand for a short time.

4.The breast milk doesn't flow and pain in the breast area:

•If you feel pain and no milk flows during expression, maybe your mammary gland is blocked.

•The block of the mammary gland may result in mastitis. Contact a physician promptly.

•The pain after suction may be caused by mastitis. We recommend to express the breast milk on time to mastitis.

5.Pain during suction:

If you experience pain when squeezing, you can relieve it in the following ways:

•Adjust the position of the massage pad so that the nipple to be exactly in the center of the funnel of the breast pump body. Like this the pad will massage your breast better and will relieve the pain during pumping.

•Use lower level of suction.

•The more regularly you use the breast pump, the faster you will get used to this way of expressing and your breast pain will gradually decrease.

• Consult your health professional/ breastfeeding advisor.

6.Milk is dropping down the silicone massage pad:

•Remove and refit again the massage pad in order to ensure it is firmly attached to the central part of the funnel.

•Try to lean slightly forward.

7.There is still breast milk that you weren't able to express it?

The mammary gland will continue to secrete milk due to its reticular structure and stimulation by elevated prolactin levels. It is normal for you not to be able to express all the milk that your gland has produced through the pump. Therefore, we recommend that you squeeze the remaining amount by hand.

8.Cracking and discoloration of the pump/ feeding bottle parts:

Avoid contact with antibacterial or abrasive detergents/ cleaners as they can damage the plastic. Combination of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations can cause plastic to crack. If this occurs, do not use the breast pump. If you clean the pump body and the breast milk containers in dishwasher, the food colorings may cause discoloring them. Keep the product out of direct sunlight as prolonged exposure may lead to discoloration of the plastic.

9.Lost or broken parts:

•For spare parts contact an authorized service center or the shop from which you have purchased the product. Do not use spare parts, which are not approved or supplied by the manufacturer.

•Do not attempt to repair or to modify the product, because this may endanger Your and Your baby's health.

•In the event of damage, contact the store from which you purchased the product immediately.

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