MANUAL BREAST PUMP

with bottle 150ml

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Product Description

Easy to use and practical, LORELLI pump allows you to regulate the intensity of squeezing.

Designed for light and convenient contact with the breast, the pump is compatible with wide neck bottles. The pump bottle is used directly for feeding.

Product Features

- Easy to use - single hand operation.
- Easy cleaning and assembly.
- Soft silicone massaging pad adjusted to breast shape.
- Comfortable, hand-adjusted handle.

Cleaning and Sterilizing

Before first use, clean the product.

Before first use place in boiling water for 5 minutes. This is to ensure hygiene.

Before and after each use wash thoroughly by using bottle brush with bottle cleaning agents and wash by using clean water.

Squeeze the teat to ensure any residual milk has been removed.

Sterilize by immersing in a feeding bottle sterilizing solution, according to the manufacturers instructions.

Alternatively, boil in water for at least 5 minutes or use a steam sterilizer.

Don't clean or allow to come into contacts with solvents or harsh chemicals. Damage could result.

Clean before each use.

For your child's safety and health

WARNING! Always use this product with adult supervision.

Before and after feeding:

- Never leave the child unattended during feeding.
- Never use feeding teats as a soother.
- Squeeze the nipple to remove any water or milk left.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.

NEVER PUT A SEALED BOTTLE IN A MICROWAVE STERILIZING UNIT

- Do not microwave directly. Use microwave sterilizing unit only.

The outside of the bottle will feel cooler than the food therefore shake well to ensure even heat distribution and always check the temperature of the food before offering to your baby.

How to Store Your Expressed Breastmilk

Breastmilk should be stored in a clean, sealed container. Do not store your breastmilk in a freezer door. The door tends to be the warmest part of the freezer because of opening and closing.

Thawing

Thaw frozen milk in the refrigerator or place the container under cool running water, gradually warming the water until the milk is thawed. Thawed milk can be kept in a refrigerator for up to 24 hours. Do not refreeze milk once it is thawed

Gently swirl the container of thawed milk before feeding to baby to mix the layers that have separated. Breastmilk is not homogenized and the "cream" will rise to the top of the bottle. Breastmilk can vary in color and consistency depending upon the time of the day that the milk was expressed, what you have eaten, and the age of the baby at the time of pumping. Do not shake the bottle vigorously to mix the layers as doing so can damage the live components of your milk.

Do not microwave human milk! Microwaving alters the composition of the milk. Do not thaw breastmilk in very hot or boiling water.

- Adapter cap can be used with wide neck bottles.
- Wide neck for easier filling and cleaning.
- Wide neck feeding bottle with silicone teat.
- Carefully made to the highest safety standards.

Instructions for safe feeding

Breast milk is best for you baby. When breast feeding is not possible, this bottle may be used to give expressed breast milk or prepared infant formula to your baby.

- Always wash your hands before preparing the food and before feeding the baby.
- Inspect before each use and pull the feeding teat in all directions. Throw away at the first signs of damage or weakness and replace it with new one.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant ("sterilizing solution") for longer than recommended, as this may weaken the teat.
- Keep all components not in use out of reach of children.
- Never dip the nipple in sweet substances! This may cause tooth decay. Replace the nipple each one or two months to keep its properties and safety!

Recommendations:

Drawing milk from breasts can be done for many reasons:

- If your breasts produce too little milk and you want to increase the production: using breast pump stimulates feeding process, which leads to lactation increase.
- After breast feeding when you still have milk.
- When you notice sudden milk overfilling in your breasts (milk lots), pumping it out will loosen them and give you a relief.
- If your nipple goes inside the overfilled breast drawing a little milk before the actual feeding time will soften the areola of the mamma (edging) and bring the nipple forth, by which it will make it easier to grab.

When you feel sudden overfilling between feeding times, you can deal with it without waking up the baby.

No Milk Expressed: If you are not successful in expressing any milk, try the following:

- Make sure that all the parts have been properly dried and assembled and that proper suction is created.
- Try to relax as successful pumping is a learned art and may take several practice attempts to get started.

Pain When Pumping - If you are experiencing pain when expressing milk, try the following: You may be pumping too hard. It may not be necessary to use all of the suction that the pump can generate. Try depressing the handle half way and remember the 2-3 second suggested rhythm.

Breastmilk Storage Guidelines

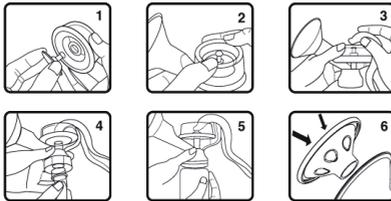
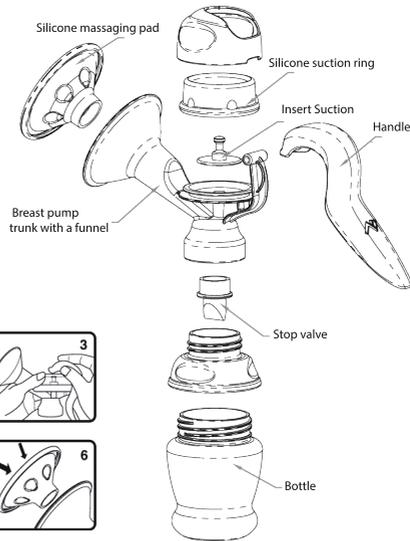
Where	Temperature	Time
At room temperature (fresh milk)	66 ° to 78 ° F	4 hours (ideal) up to 6 hours (acceptable)*
	(19 ° to 26 ° C)	(Some sources use 8 hours)
In a refrigerator	<39°F (>4°C)	72 hours (ideal) up to 8 days (acceptable)**
	-0.4° to -4°F (-18° to -20°C)	6 months (ideal) up to 12 months (acceptable)

* The preference is to refrigerate or chill milk right after it is expressed.

** Eight days is acceptable, ideally collected in a very clean, careful way.

Assembly:

- A** Silicone massaging pad
- B** Funnel
- C** Silicone suction ring
- D** Insert suction
- E** Stop valve
- F** Handle
- G** Bottle

**How to use your breast pump:**

Before starting to use the pump, wait until your milk supply and breastfeeding schedule are well established. This is normally at least 3-4 weeks after birth.

1. Prepare clean, sterilized and ready to use breast pump.
2. Wash hands with soap carefully.
3. Warm the breast before the process: bath towel or warm shower.
4. Massage breast with circular moves and slightly rub the nipple, just like during breast self-examination. This stimulates milk production as well as its flow.
5. Place the funnel of the pump on the breast so that the nipple gets inside it and no air is coming through. There must be vacuum for the pump to work properly. If the inside of the funnel gets damp, the nipple will freely translocate.
6. Draw milk for about 5 minutes, stop, massage and rub the nipple, then continue pumping for another 5 minutes. Milk should be drawn for 10 minutes from each breast. When using only one device, pumping should be done by turns-changing breasts. Longer pumping will not produce more milk. Shorter, yet more frequent pumping stimulates milk production and better breast emptying. The more practice the more quantity of milk. Handle pressing level - sucking power and frequency can be individually adjusted.

7. It is advised to drink a glass of water during the procedure.

8. Drawn milk should be placed in plastic bottles.

- Milk obtained after every session should be placed in separate bottles labeled with date and time of the procedure.
- Drawn milk can be stored for 10-12 hours in a refrigerator.

WARNING! Should you have any doubts whether you store your milk properly, it will be safer for your baby if you contact the doctor or feed the baby with freshly drawn milk.

9. Milk should begin to flow after the first few strokes. Do not worry if your milk does not flow immediately. Relax and continue pumping.

Warning! Do not continue pumping for more than five minutes at a time if no result is achieved. Try expressing at another time during the day. Should pumping become very painful or uncomfortable, you should stop using the pump and consult your doctor.

kit includes:

- 1 manual breast pump with silicone massaging pad
- 1 bottle of 150 ml for feeding and milk storage
- 1 feeding teat
- 1 protective cover

The feeding teat and the protective disk are suitable with the current breast pump.

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Didis Ltd., Bulgaria, Shumen,